



STAINLESS STEEL MOBILE SHOWER COMMODES

Attendant Propelled | Self Propelled | Tilt and Recline

USER MANUAL



Thank you for choosing this Stainless Steel Commode from our Aspire Mobile Shower Commode Range.

This product has been specifically designed to offer comfort, support and safety.

It is important that you read this user manual carefully before using your new shower commode.



This product features the CE mark, in compliance with the Medical Devices Regulation 2002 (as amended) Class 1, and the Medical Device Regulation 2017/745



This product Features the UKCA mark, in compliance with Part II UK MDR 2002 (as amended) Class 1

© Aidacare Pty Ltd 2024

No part of this user manual may be copied, distributed, reproduced, scanned or stored on any electronic database, whether in whole or in part in any form or by any means.

Due care has been taken to ensure all the information contained in this user manual is correct at the time of printing. All measurements, pictures, colours and weight capacities are to be used as a guide only. We reserve the right to modify the design or appearance of any product displayed in this user manual without prior notice.

TABLE OF CONTENTS

Symbols Used In This Manual	4
Symbols Used On The Product	4
1. Introduction	5
2. Safety Precautions/Warnings	6
3. Product Overview	8
4. Unpacking and Inspection	11
5. Assembly Instructions	15
6. Operation	19
7. Accessories	22
8. Troubleshooting	23
9. Cleaning	24
10. Service and Maintenance	24
11. Specifications	25
12. Compliance	26
13. Warranty	26
14. Manufacturer, EU Representative & Importer Details	27

SYMBOLS USED IN THIS MANUAL

The symbols below are used throughout this user manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.



WARNING

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage or malfunction. On the product, this icon is represented as a black symbol on a white triangle with a black border.



ALWAYS! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white infinity symbol on a black dot with a white border.



DO NOT! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol with a black circle and black slash.

SYMBOLS USED ON THE PRODUCT

Sticker shown below is a sample, product specifics will change per item type.



Product Code



Safe Working Load



Serial Number



Read User Manual



Importer



Caution



Country of Manufacture



Medical Device



UKCA Mark



Authorised Representative in EU



CE Mark

1. INTRODUCTION

The Aspire Mobile Shower Commode range is designed for a healthcare environment such as a facility or hospital. The commodes have a fabricated stainless steel frame and utilise plastic and PU components of various material specifications.

CLINICAL APPLICATIONS AND AREA OF USE

These commodes are appropriate for adult patient care and should only be used under the instruction of a carer or health professional where concerns exist around patient use.

KEY FEATURES

- Maximum Safe Working Load (SWL) of 200kg/31.5st or 400kg/63st (dependent on model)
- Attendant propelled and self propelled options available
- Height adjustability to accommodate different users (TIS model only)
- Stainless steel finish
- Swing back safety arms for ease of cleaning and transfer
- Soft PU seat range in open or closed variations (sold separately)
- Easy slide weight bearing platform or swing away legrest options available
- Access trough with improved over toilet clearance for easy hygiene access
- -8° – 18° angle adjustable backrest for improved seating comfort (swing away models only)
- Angle adjustable backrest for improved seating comfort
- Ultra low seat height of 590mm or down to 490mm when customised (swing away models only)

2. SAFETY PRECAUTIONS/WARNINGS

It is important that this User Manual is read in its entirety prior to using the commode. The manual is designed to ensure that users are equipped with the information required to competently and responsibly operate this commode.

It is recommended that the User Manual be kept in an easily accessible place for quick reference. If required, Aidacare product specialists can provide additional training.

All warnings and cautions within this manual should be strictly adhered to. Failure to do so could result in harm to the patient or operator.

WARNINGS highlighted throughout this manual identify possible hazards in procedures or conditions which if not followed correctly could result in serious injury or fatality.

CAUTIONS highlighted throughout this manual identify possible hazards in procedures or conditions, which if not followed correctly could result in equipment damage or failure.

INTENDED USE

- This product is specifically designed for individuals who, due to a wide variety of reasons, have limited mobility or require assistance when toileting and bathing. This product should be used as a tool to assist such users, with the guidance of a trained healthcare professional or care worker.
- The clinically qualified person responsible should consider the age, size and condition of the user before using the commode.
- The maximum user weight is 200kg / 31.5st or 400kg / 63st depending on the model chosen.
- **Attendant Propelled** Mobile Shower Commodes are able to be propelled by an attendant with the push handle located on the top of the backrest. This Mobile Shower Commode is designed for indoor use **ONLY** and should **NOT** be used outdoors under any circumstances. This product is suitable for **INDOOR** wet areas such as bathrooms and showers but is **NOT** suitable for use within or around pools.
- **Self Propelled** Mobile Shower Commodes are able to be propelled by a user with the self propelling wheels or by attendant with the push handle located on the top of the backrest. This Mobile Shower Commode is designed for indoor use **ONLY** and should **NOT** be used outdoors under any circumstances. This product is suitable for **INDOOR** wet areas such as bathrooms and showers but is **NOT** suitable for use within or around pools.



WARNING

WARNING! Swing Away Footplate models are not intended to bear the weight of an occupant through its footplates. **DO NOT** stand on the footplates under any circumstances.



ALWAYS check with your healthcare professional prior to using this commode.



ALWAYS apply the brakes when the commode is stationary.



ALWAYS ensure brakes remain locked when transferring into or out of the commode.



ALWAYS ensure that the legrests are swung away when transferring into and out of the commode (swing away models only).



ALWAYS ensure that attendant and patient limbs are free from potential entrapment areas before operating any functions.



DO NOT stand on the product.



DO NOT use outdoors.



DO NOT use this product without reading the user manual in its entirety.



DO NOT use if defective.



DO NOT make any changes or modifications to this product without consultation from Aidacare.

3. PRODUCT OVERVIEW

The Aspire Mobile Shower Commode range is designed specifically for Care and Hospital environments. Please take time to familiarise yourself with key components shown below:


SWING AWAY LEGREST MODELS



Self Propelled Wheels
(selected models only)



Swing Back Safety Arms

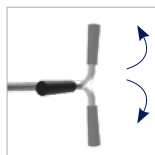

Adjustable Safety Arms



Access trough with improved over toilet clearance for easy hygiene access



Adjustable Height Legrests



Swing Away Legrests



Soft PU Seat
(Sold Separately)



4 x Stainless Steel Lockable Castors




WEIGHT BEARING PLATFORM MODELS



Self Propelled Wheels
(selected models only)



Swing Back Safety Arms

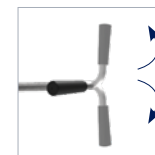

Adjustable Safety Arms



Access trough with improved over toilet clearance for easy hygiene access



Easy-Slide Weight Bearing Platform



Soft PU Seat
(Sold Separately)




4 x Stainless Steel Lockable Castors



4. UNPACKING AND INSPECTION

Aspire Mobile Shower Commodes come packaged in one box for maximum protection. Check box to ensure no components are missing and contact **Aidacare**.

TILT & RECLINE MODELS

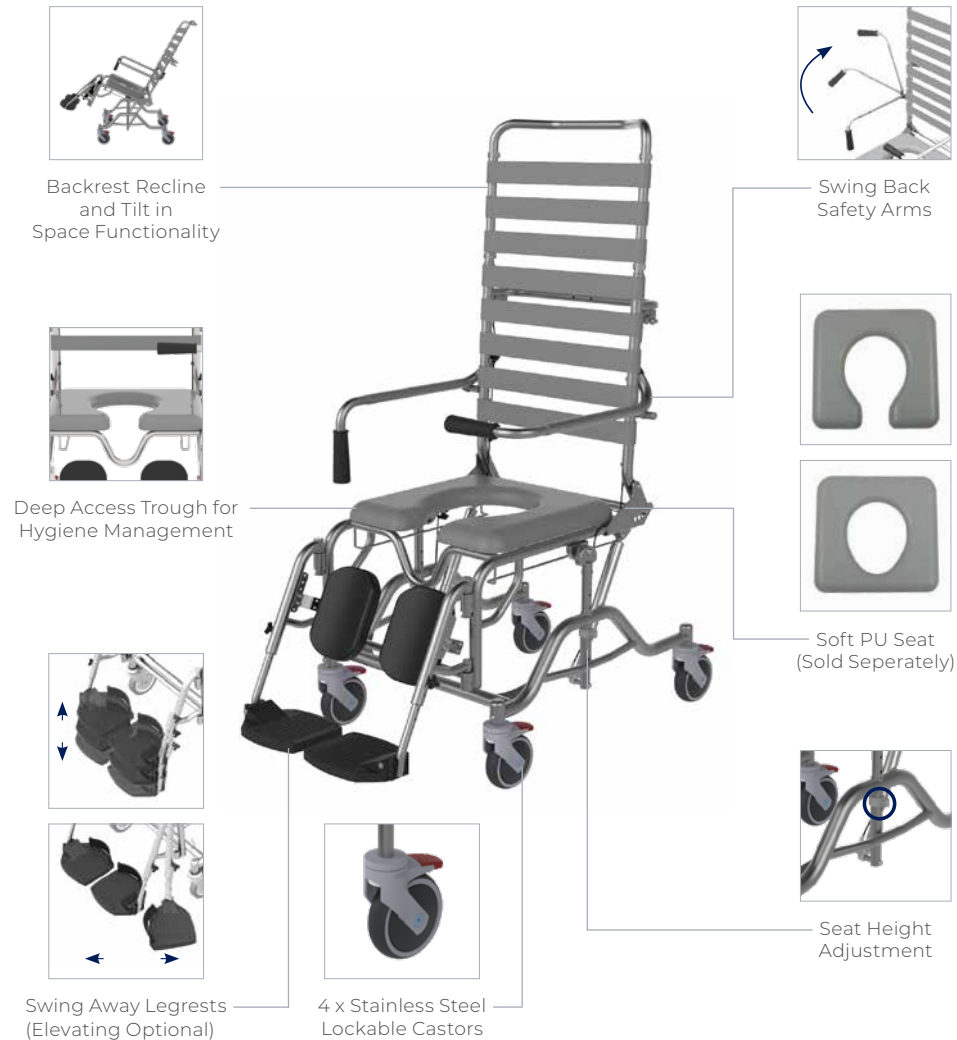


SWING AWAY LEGREST MODELS

IN CARTON



SOLD SEPARATELY

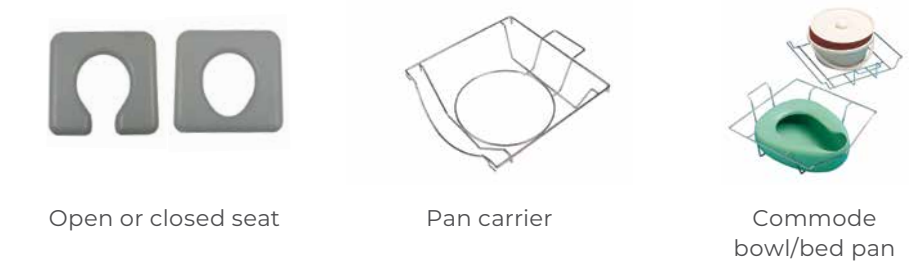

WEIGHT BEARING PLATFORM MODELS

IN CARTON



Commode frame with backrest, safety arms, locking castors



User manual (in plastic bag)



Seat mounting hardware (in plastic bag)

SOLD SEPARATELY



Open or closed seat



Pan carrier



Commode bowl/bed pan

TILT & RECLINE MODELS

IN CARTON



Commode frame with backrest, safety arms, locking castors



Swing away legrests (pair)



User manual (in plastic bag)



Seat mounting hardware (in plastic bag)

SOLD SEPARATELY



Open or closed seat



Pan carrier



Commode bowl/bed pan

SELF PROPELLING MODELS


IN CARTON



Commode frame with backrest, safety arms, locking castors



Swing away legrests (pair)

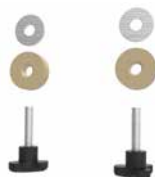

Seat mounting hardware (in plastic bag)



User manual (in plastic bag)



Self propelling kit (wheels, brakes and anti-tippers)

SOLD SEPARATELY



Open or closed seat



Pan carrier



Commode bowl/bed pan

5. ASSEMBLY INSTRUCTIONS

SWING AWAY LEGREST MODELS



Step 1
Remove commode from box and clear away all packaging materials.



Step 2
Detach screws that are fitted into backrest, swing arms and backrest into position and secure in place with screws.



Step 3
Attach legrests into lugs mounted on commode frame **1** and tighten with screws underneath top lug **2**



Step 4
Using seat mounting hardware, affix the seat **3** (sold separately) to the top of the frame and screw in place until tight **4**



Step 5
Slide in pan carrier and bowl (sold separately).

WEIGHT BEARING PLATFORM MODELS




Step 1
Remove commode from box and clear away all packaging materials.



Step 2
Detach screws that are fitted into backrest, swing arms and backrest into position and secure in place with screws.



Step 3
Using seat mounting hardware, affix the seat (1) (sold separately) to the top of the frame and screw in place until tight (2).

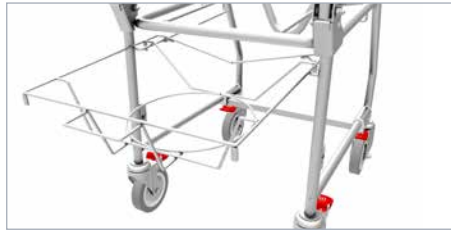


Step 4
Slide in pan carrier and bowl (sold separately).

TILT & RECLINE MODELS



Step 1
Remove commode from box and clear away all packaging materials.

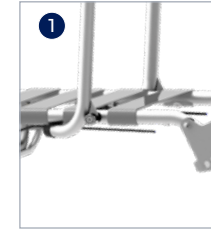


Step 2
Raise backrest into desired position (1), it will click into place when secure. Slide in top frame and secure with C Clips (2).



Step 3
Attach legrests into lugs mounted on commode frame (3) and tighten with screws underneath top lug (4).



Step 4
Using seat mounting hardware, affix the seat (5) (sold separately) to the top of the frame and screw in place until tight (6).

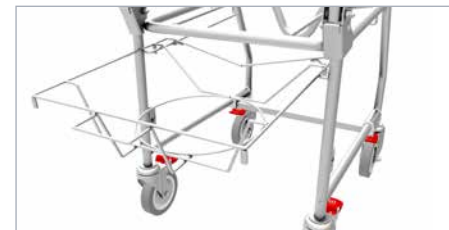

Step 5
Slide in pan carrier and bowl (sold separately).

SELF PROPELLING MODELS




Step 1

Remove commode from box and clear away all packaging materials.



Step 2

Detach screws that are fitted into backrest, swing arms and backrest into position and secure in place with screws.

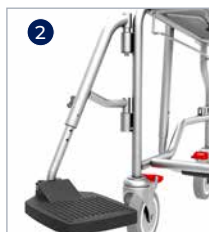

Step 3

Attach legrests into lugs mounted on commode frame ① and tighten with screws underneath top lug ② (swing away legrest models only).



Step 4

Using seat mounting hardware, affix the seat ③ (sold separately) to the top of the frame and screw in place until tight ④



Step 5

Slide in pan carrier and bowl (sold separately).



Step 5

Remove locking pin from frame ⑤, insert through centre of wheel ⑥, mount brake with clamp onto each side of frame ⑦ detach rear castors and replace with anti-tippers. ⑧

6. OPERATION

SEAT INSTALLATION (SOLD SEPARATELY)

- Place seat onto top of frame and align along guides ①
- Using tri-wheel mounting kit (supplied with commode) screw the tri-wheels firmly into place until seat is secured onto the frame ②



CASTOR BRAKE ACTIVATION – ATTENDANT PROPELLED MODELS

- To engage, push lever down with your foot ①
- To disengage, push the top end of the lever with your foot ②



HAND BRAKE ACTIVATION – SELF PROPELLED MODELS

- Push down and forward on the black hand levers on both sides of the commode to lock rear wheels in place ①
- Pull back on the hand lever to disengage brakes ②




LEGREST ADJUSTMENT (SWING AWAY LEGREST MODELS ONLY)

- Lift and swing the legrests to desired position ①
- If the user wishes to vacate the chair, swing the footrests outwards for clearance. They can be completely removed by removing locking screw and lifting upwards ②




BACKREST ANGLE ADJUSTMENT (ALL MODELS EXCEPT 600MM WEIGHT BEARING AND 460/530MM TIS)

- Backrest adjustment is facilitated by re positioning the offset washer, located on plate that connects the backrest to the seating support ①. There is 10° of angle adjustment available (8° or 18°).
- Using a 5mm Allen key and 10mm Shift lever, loosen two bolts ② on either side of the commode.
- Rotate the offset washer on either side of the commode ③, adjust the backrest to the desired position, lock in place on either side of the commode.

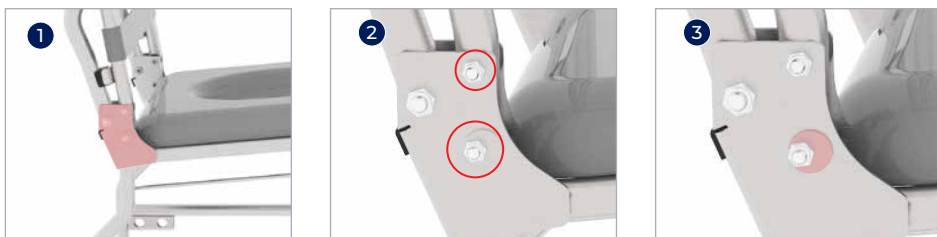

PAN CARRIER & BED PAN INSTALLATION (SOLD SEPARATELY)

- Holding the handle of pan carrier, insert the front end into the mounting rails at the rear of the commode ①
- Place commode bowl or pan into pan carrier before sliding pan carrier underneath seat ②

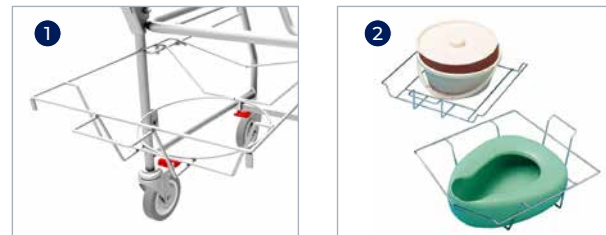

USING SAFETY ARMRESTS

- Swing armrests to desired position and ensure spring pin locks into place ①



7. ACCESSORIES




Open Seat

BTC066600 – 460mm
BTC066750 – 530mm
BTC066800 – 600mm



Closed Seat

BTC066700 – 460mm

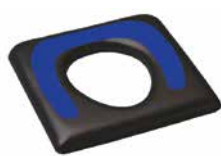

Hand Made Pressure Reducing Seat – 460mm

Multiple sizes available, refer to brochure for more details



Commode Bowl Carrier

BTC066400



Oxygen Bottle Carrier

BTC067200



Armrest – Padded Offset

BTC067000



Commode Bowl and Bed Pan

BTC081610 – Bowl and Lid
BTT151800 – Bed Pan



Adjustable Pelvic Strap

BTC067100



Self Propelling Conversion Kit

BTC082900



Elevating Legrests

BTC067110 – Left
BTC067115 – Right



Adjustable Thoracic Support

WCA867010



Advanced Headrest

BTC067050



Stump Support

BTC067105



Adjustable IV Pole

BTC067103

8. TROUBLESHOOTING

Fault	Cause	Solution
Difficulty in removing/adjusting footrests	Build up of material in receivers	Remove footrests, clean receivers and footrest mounts
Difficulty adjusting the seat height	Build up of material in seat frame adjustment point	Remove upper from lower frame and clean thoroughly
Difficulty in maneuvering castors	Build up of dirt between fork and castor	Clean castors thoroughly
Difficulty in adjusting commode seat	Build up of material between frame and seat	Remove seat and clean thoroughly

Please contact **Aidacare** if problems persist.

9. CLEANING

WARNING

Note: It is strongly recommended that protective clothing be used when cleaning.

- Frame and metal components – Always keep the commode clean for greater comfort and long-lasting performance. Clean stainless steel components with water and a mild detergent. Wipe dry.
- Brakes – The braking effect depends on the state of the tyre tread. Excessive dirt build-up may negatively affect the application of the brakes. If necessary, clean tyres with a small brush to remove dust and waste.
- Seat and backrest padding – Easy clean material that can be washed with water and neutral soap/detergent.
- Commode Bowl – Remove and clean with water and a neutral soap/detergent.

CAUTION

Do not use abrasive Hypocarbonate or Phenolic compounds or pads.

10. SERVICE AND MAINTENANCE

To prolong the working life of the commode and for trouble free operation it is recommended that routine inspections should be performed by a person who is suitably qualified and well acquainted with the design, use and care of Aspire Mobile Shower Commodes.

If inspection reveals that the safety of the commode is compromised in any way, it should be IMMEDIATELY WITHDRAWN FROM SERVICE, tagged as such and the owner notified, until repair or replacement has occurred.

INSPECTION:

A visual and functional inspection should be performed monthly and every time the commode changes user.

- Visually inspect mechanical components: Can you see damage, chips or deformation to the frame, armrests, seat, backrest, legrests or commode bowl?
- Are the castors, wheels and brakes functioning as they should?
- Are all frame securement hand wheels present and tightened?

11. SPECIFICATIONS

	Seat Width mm inches	Overall Width mm inches	Overall Height mm inches	Backrest Height mm inches	Armrest Height mm inches	Anterior Seat Height mm inches	Posterior Seat Height mm inches	SWL kg stone
BTC066000 Attendant Propelled – WB Platform	460 18	535 21	995 39	305 12	225 9	590 23	560 22	200 31.5
BTC066200 Self Propelled – WB Platform	460 18	665 26.2	995 39	305 12	225 9	590 23	560 22	200 31.5
BTC066050 Attendant Propelled – WB Platform	530 21	605 24	995 39	305 12	225 9	590 23	560 22	200 31.5
BTC066055 Self Propelled – WB Platform	530 21	735 29	995 39	305 12	225 9	590 23	560 22	200 31.5
BTC061400 Attendant Propelled – WB Platform	600 24	725 28.5	995 39	300	290 11.5	590 23	560 22	400 63
BTC065900 Attendant Propelled – Swing Away Footrests	460 18	535 21	895–995 35–39	305 12	225 9	490–590 19–23	560 22	200 31.5
BTC066100 Self Propelled – Swing Away Footrests	460 18	665 26.2	895–995 35–39	305 12	225 9	490–590 19–23	560 22	200 31.5
BTC061300 Attendant Propelled – Swing Away Footrests	600 24	675 26.5	995 39	305 12	225 9	490–590 19–23	560 22	200 31.5
BTC061305 Self Propelled – Swing Away Footrests	600 24	805 32	995 39	305 12	225 9	490–590 19–23	560 22	200 31.5
BTC066030 Attendant Propelled – Tilt & Recline	460 18	560 22	1315–1485 52–58	900 35.5	275 11	460–630 18–25	460–620 18–24.5	250 39
BTC066040 Attendant Propelled – Tilt & Recline	530 21	630 25	1315–1485 52–58	900 35.5	275 11	460–630 18–25	460–620 18–24.5	250 39
BTC066060 Attendant Propelled – Swing Away	530 21	605 24	995 39	305 12	225 9	490–590 19–23	460–560 22	200 31.5
BTC066065 Self Propelled – Swing Away	530 21	735 29	995 39	305 12	225 9	490–590 19–23	460–560 22	200 31.5

*Swing away model are only adjustable when customised

12. COMPLIANCE

This Commde range is manufactured in ISO compliant production facilities in PR China to strict quality control standards.

Aspire Commodes are manufactured and independently tested to the market-relevant compliance standards.

Labels on the compliance address compliance requirements and users should familiarise themselves with these prior to use.

13. WARRANTY

Aspire Mobile Shower Commodes come with a five year corrosion resistance warranty on the external frame surface. If the product surface becomes damaged this will void the warranty. A general one year warranty also applies covering faulty workmanship or defective materials. This warranty is valid for twelve months from the original purchase date, and to the original customer.

Aidacare does not warrant against faults caused by incorrect use, unauthorised modification or any situation that could not be deemed as fair wear and tear.

The warranty on this product is a back to base warranty and does not cover any freight costs pertaining to the return of any items under warranty. For further information contact **Aidacare**.

Aidacare will not guarantee the safety and the correct functioning of the product in the case that any of the original components have been changed or modified and or replaced with non-original Aspire approved parts. If any faults are detected upon receiving the product please phone **Aidacare** to arrange a replacement. Any faults that are detected during normal use should be reported to **Aidacare** immediately to determine if warranty conditions apply and organise repair and/or replacement as applicable.

For servicing, preventative maintenance and any other questions regarding this or any other **Aidacare** product please contact Aidacare.

Spare parts list available on request.

14. MANUFACTURER, UK REPRESENTATIVE AND IMPORTER DETAILS

If you have any questions, you can contact:




Manufacturer:

Aidacare Pty Ltd

ABN 40 134 398 833

Building 3A, 1 Moorebank Avenue,
Moorebank NSW 2170
Australia

1300 133 120 | product@aidacare.com.au

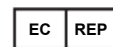

EU Representative:

European Device Solutions (Ireland) Ltd

Stratton House, Bishopstown Road,
Cork T12 Y9TC Ireland
info@europeandevicesolutions.eu



Importer:

Aidacare Ltd

Registration 13785408

Arcadia House, Maritime Walk, Ocean Village,
Southampton SO14 3TL United Kingdom

01622 541235 | info@aidacare.co.uk

AIDACARE PTY LTD

Building 3A, 1 Moorebank Avenue,
Moorebank NSW 2170
Australia

1300 133 120 | aidacare.com.au

AIDACARE LIMITED

Arcadia House, Maritime Walk,
Ocean Village, Southampton SO14 3TL
United Kingdom

01622 541235 | aidacare.co.uk

MADE IN PRC

ASUM22.43 080224

